



t+diabetes – Mobile Phone Diary Questions

Configuration

The list of questions presented to a patient is configured on the website by a clinician. These options can be set on an individual or clinic wide basis.


Clinic Configuration on Website

* User web language:	English (United Kingdom) ▾
* User phone language:	English (United Kingdom) ▾
* Timezone:	Europe/London ▾
* Weight measurement units:	Kg ▾
* Blood glucose measurement units:	mmol/L ▾

Patient Configuration on Website

* Are you taking insulin?:	<input checked="" type="radio"/> Yes <input type="radio"/> No	6
* Do you monitor your blood pressure?:	<input checked="" type="radio"/> Yes <input type="radio"/> No	3

Flow of Diary Questions

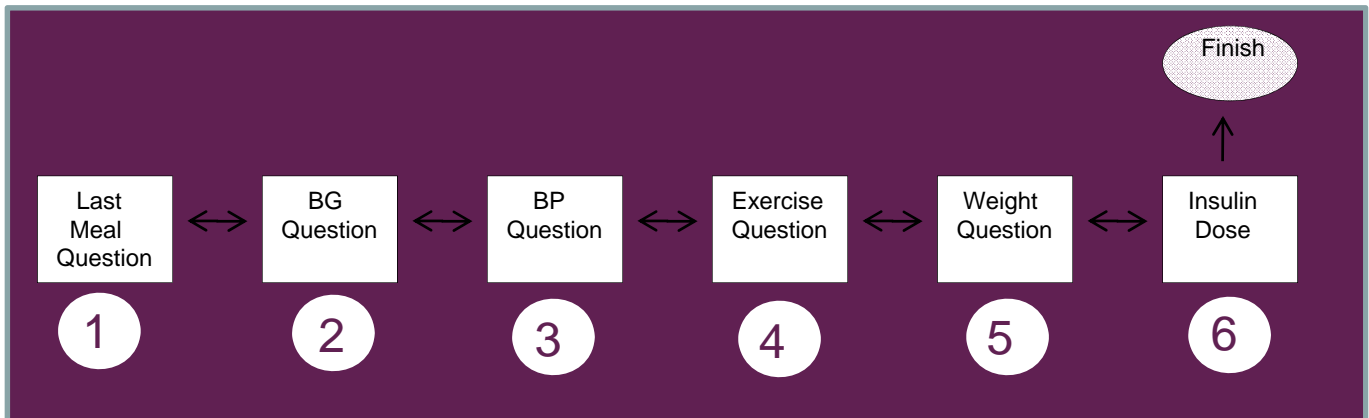

The blood pressure screen (3) and insulin dose screen (6) will be shown depending on the configuration for the patient.

Diary Screens:

1

Last meal
How long is it since you last had a meal?

0-1 hour
1-2 hours
2-4 hours
4-8 hours
More than 8 hours

Back Next

2

B.G. Reading
Enter your Blood Glucose Reading (0.0 - 38.0)

mmol/L

Take a reading, then press BT for Bluetooth link

Back BT Next

3

Blood Pressure
Enter your BP (10 - 250)

Systolic
mmHg

Diastolic
mmHg

Press BT, then take a reading for Bluetooth link

Back BT Next

4

Exercise
How much exercise will you do today?

Skip
Less than usual
About average
More than usual

Back Next

5

Weight
Enter your Weight (5.0 - 200.0)

Kg

Back Next

6


Insulin
BG Reading = 8.0 mmol/L

Enter Dose (0 - 200)

Long
Short
Mixed

Back Next

Feedback Screens:

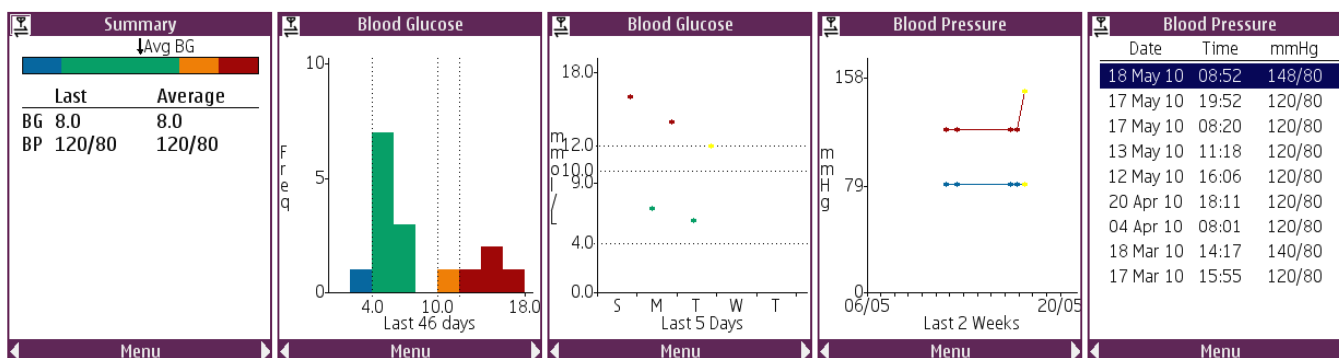

UK: 174 Milton Park, Abingdon, OX14 4SE, UK Tel 0800 389 3205
 USA: 6320 Quadrangle Drive, Suite 3270, Chapel Hill, NC27517, USA Tel 866 424 6744
 Email: info@obsmedical.com