



What you get:

Product Code: 7120

1. A WristCare Home System, consisting of everything you need for one person including a wrist unit and base unit, pre-programmed and ready for connection to the telephone line. The system is preconfigured with the following features active:
 - ✓ System alarms
 - ✓ Manual alarm
 - ✓ Non-Compliance alarm
 - ✓ Inactivity alarm
 - ✓ User Out alarm
2. A wireless Device Interface which incorporates a door monitor.
3. The partnership and support of Vivatec in your Telecare implementation. Please see the WristCare Implementation Plan literature for details.

Explanation of alarms:

| | |
|----------------------|---|
| System alarms | Alarms to notify the alarm service in the case of mains power failure or disconnection, low batteries in wrist unit, low back up battery in base unit. |
| Manual alarm | The user has pressed the manual button on the wrist unit. |
| Non-Compliance alarm | The user has taken off their WristCare and cannot summon help. The alarm gives the alarm service the opportunity to prompt the user to put it back. It also provides an objective measure of compliance. |
| Inactivity | The user is unusually inactive and appears to be unconscious. This may be the result of a fall, or any other acute event. |
| User Out Alarms | Based on a combination of the door being opened (detected wirelessly using the Device Interface) and the user going out of range of the base unit. Installation of the Device Interface requires 4 holes to be drilled (2 on the door and 2 on the frame) and 4 screws to be used (supplied). |

21/02/2007